


# Vascular Tourniquet with plastic stylet

Revision date: 10.07.2018

## Description

Deep vascular occlusion system using loops (not included).

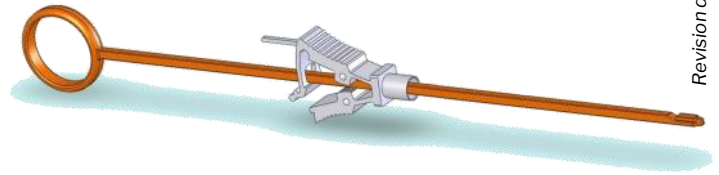

## Use

The device permits rapid, easy and safe occlusion of blood vessels. After positioning the loop with a double passage around the blood vessel (the loop is not included with the device and must be chosen according to the condition that the case requires), the two ends will be anchored at the stylet's terminal.

Once the stylet has been pulled to the outside of the occlusion tube, it is possible to place the loop in traction so to obtain the occlusion pressure desired. At the conclusion of this phase it is sufficient to close the proximal clamp to fix the loop in place.

Using this device, the surgeon can maintain complete control on the blood vessel, which also permits the precise administering of heparin locally. The quick release clamp allows rapid control of blood flow.

Various models are available with occlusion tubes in silicone or PVC, with coloured radiopaque line to distinguish between different blood vessels (various arteries and/or veins).

All occlusion tubes have non traumatic rounded tips.

## Specifications

- Clamp and stylet made of polyamide
- Occlusion tube made of PVC or silicone, int. diameter 4mm, ext. diameter 6mm.
- Usable stylet length: 160mm.
- Usable tube length: 5cm., 10cm., 15cm.,
- Double Peel packaging, coated medical paper PET-PE
- Latex and phthalate free
- Sterilized by gamma rays



CLASS IIa

RULE 7

STERILE R

REF TOUR-101

1 stylet with clamp + 1 PVC tube  
length 5cm. with blue radiopaque line

REF TOUR-102

1 stylet with clamp + 1 silicone tube  
length 5cm. with white radiopaque line

REF TOUR-103

1 stylet with clamp + 1 PVC tube  
length 10cm. with blue radiopaque line

REF TOUR-104


1 stylet with clamp + 1 silicone tube  
length 10cm. with white radiopaque line

REF TOUR-105

1 stylet with clamp + 1 PVC tube  
length 15cm. with blue radiopaque line

REF TOUR-106

1 stylet with clamp + 1 silicone tube  
length 15cm. with white radiopaque line

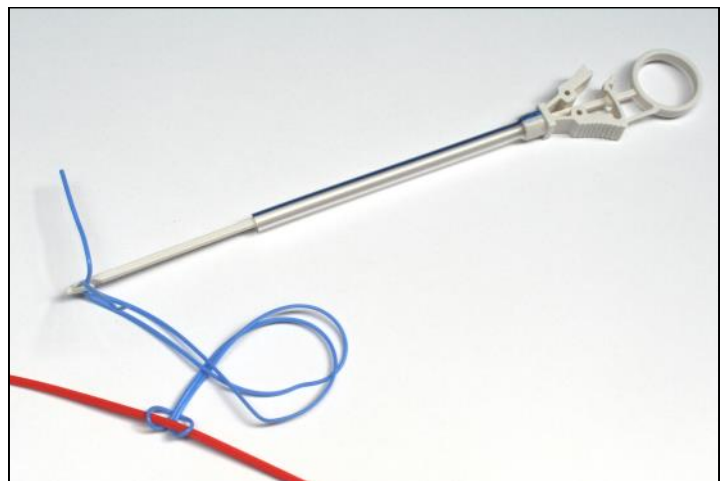

REF TOUR-301

1 stylet with clamp + 1 silicone tube length 5cm. with white radiopaque line + 1 loop 90mm. Maxi in silicone

REF TOUR-302

1 stylet with clamp + 1 silicone tube length 10cm. with white radiopaque line + 1 loop 90mm. Maxi in silicone

Packaging: 6 pcs.

Box of 36 pcs.

Manufactured by:



Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy  
Tel. +39 051.817902 +39 051.817850 Fax +39 051.817845  
www.simalia.it www.medeurope.eu  
E-Mail: info@medeurope.eu

Distributed by:



# Vascular Tourniquet with stainless steel stylet

## Description

Deep vascular occlusion system using loops (not included).

## Use

The device permits rapid, easy and safe occlusion of blood vessels. After positioning the loop with a double passage around the blood vessel (the loop is not included with the device and must be chosen according to the condition that the case requires), the two ends will be anchored at the stylet's terminal.

Once the stylet has been pulled to the outside of the occlusion tube, it is possible to place the loop in traction so to obtain the occlusion pressure desired. At the conclusion of this phase it is sufficient to position a surgical clamp on the tube body to fix the loop in place.


Using this device, the surgeon can maintain complete control on the blood vessel, which also permits the precise administering of heparin locally.

The kit includes a stylet of stainless steel and 5 occlusion tubes made of PVC with a radiopaque line of different colours to distinguish between different blood vessels (various arteries and/or veins).

All occlusion tubes have non traumatic rounded tips.

## Specifications

- Stylet made of stainless steel AISI 316
- Occlusion tube made of PVC, int. diameter 3mm., external diameter 5mm.
- Usable stylet length: 23cm.
- Usable tube length: 18cm.
- Double Peel packaging, coated medical paper PET-PE
- Latex and phthalate free
- Sterilized by gamma rays

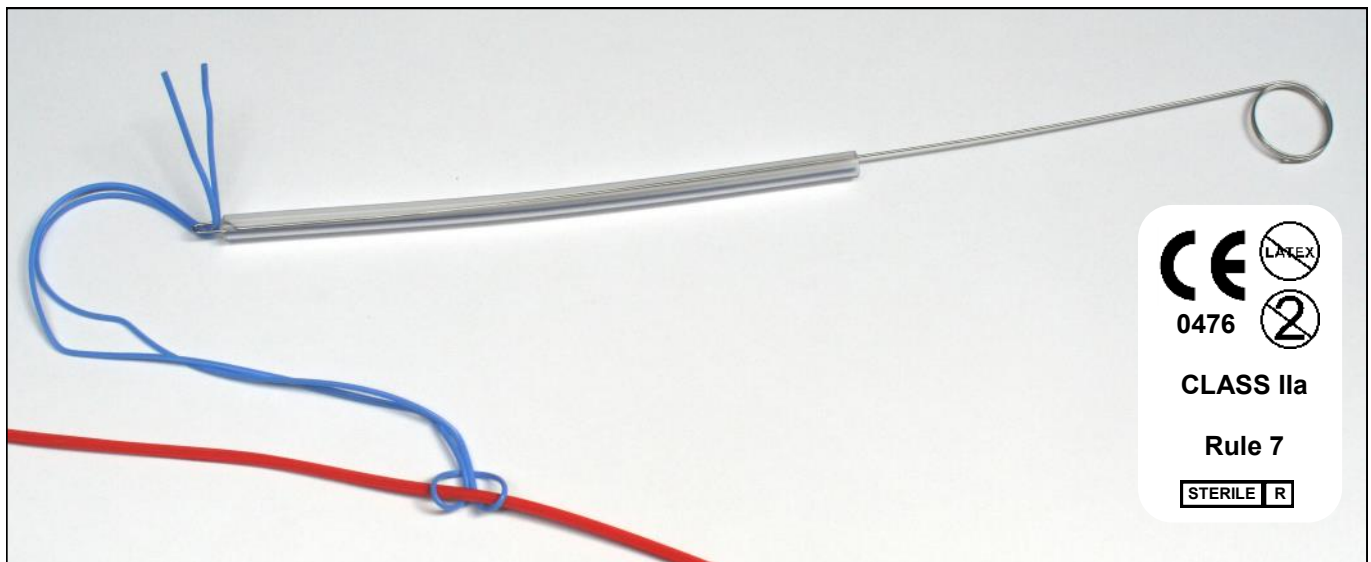

REF TOUR-002

1 stylet made of stainless steel  
 1 PVC tube with blue radiopaque line  
 1 PVC tube with red radiopaque line  
 3 PVC tubes with white radiopaque line

Packaging 6 pcs.

Box of 36 pcs.

Manufactured by:



Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy  
 Tel. +39 051.817902 +39 051.817850 Fax +39 051.817845  
 www.simitalia.it www.medeurope.eu  
 E-Mail: info@medeurope.eu

Distributed by:

