


SIM MICRO CANNULA “Slow Reaction”

Steel suction cannula with suction control handle

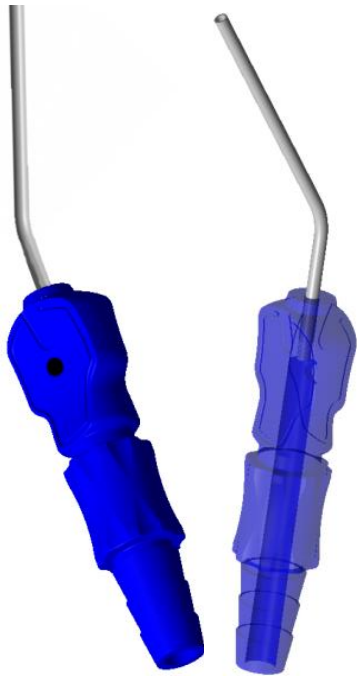

Why?

- Because reusable suction cannulae with fine lumens are very difficult to clean
- Because disposable suction cannulae are recommended for very high risk procedures, especially neuro and ophthalmic

Characteristics

- Aspiration control hole has a diameter of 1,5 mm., allowing the surgeon a slower aspiration response. Whenever the hole is opened the aspiration on the tip will slowly decrease. Even with a very high aspiration pressure the cannula makes only a low amount of noise.
- Complete line and specials can be easily produced following your indications
- Rounded edges, atraumatic distal end of lumen
- Non blocking design
- Strong enough to use as a probe. Cannulae are made of Steel AISI 304
- Single angle
- Internal diameter is uniform over the entire length of the cannula
- Suction control handle made of copolymer styrene-butadiene (ABS).
- Matches all commonly used connection tubes
- Colour code for prompt recognition. Latex free
- Sterilised by gamma radiation. Single sterile blister packaging



CLASS IIa
RULE 6

STERILE R



| Code | Internal Diameter mm. | External Diameter mm. | CH | Cannula length mm. | Handle colour | Pcs. per BOX | Applications |
|------------|-----------------------|-----------------------|----|--------------------|---------------|--------------|--|
| CSSR07X70 | 0,43 | 0,7 | 2 | 70 | BLACK | 60 | ENT |
| CSSR07X90 | 0,43 | 0,7 | 2 | 90 | BLACK | 60 | ENT |
| CSSR10X70 | 0,7 | 1 | 3 | 70 | BROWN | 60 | ENT |
| CSSR10X90 | 0,7 | 1 | 3 | 90 | BROWN | 60 | ENT |
| CSSR14X70 | 0,96 | 1,4 | 4 | 70 | WHITE | 60 | ENT |
| CSSR14X90 | 0,96 | 1,4 | 4 | 90 | WHITE | 60 | ENT |
| CSSR20X80 | 1,45 | 2,0 | 6 | 80 | GREEN | 60 | ENT, Plastics, Otological Surgery & outpatients |
| CSSR20X130 | 1,45 | 2,0 | 6 | 130 | GREEN | 60 | General, Plastics, Obs & Gynae, Paediatrics, Neuro, Oral & Max Fax |
| CSSR20X160 | 1,45 | 2,0 | 6 | 160 | GREEN | 60 | General, Plastics, Obs & Gynae, Paediatrics, Neuro, Oral & Max Fax |
| CSSR30X80 | 2,54 | 3,0 | 9 | 80 | BLUE | 60 | ENT, Plastics |
| CSSR30X130 | 2,54 | 3,0 | 9 | 130 | BLUE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |
| CSSR30X160 | 2,54 | 3,0 | 9 | 160 | BLUE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |
| CSSR40X80 | 3,0 | 4,0 | 12 | 80 | WHITE | 60 | ENT, Plastics |
| CSSR40X130 | 3,0 | 4,0 | 12 | 130 | WHITE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |
| CSSR40X160 | 3,0 | 4,0 | 12 | 160 | WHITE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |

Manufactured by:




Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy
Tel. +39 051.817902 +39 051.817850 Fax +39 051.817845
www.simitalia.it www.medeurope.eu
E-Mail: info@medeurope.eu

Distributed by:



SIM MICRO CANNULA “Slow Reaction”

Steel suction cannula with suction control handle and plastic cleaning stylet

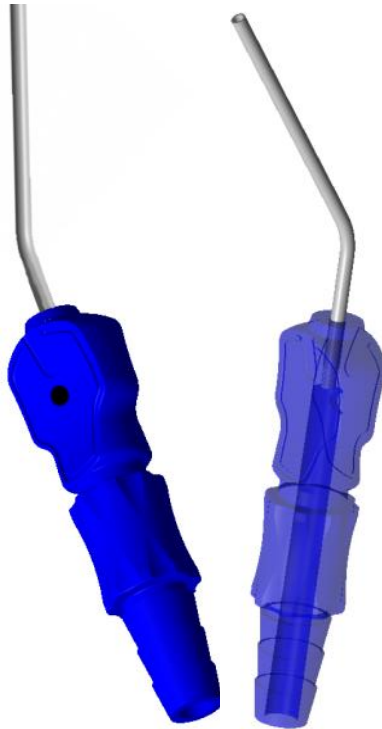

CLASS IIa

RULE 6

STERILE R



Why?

- Because reusable suction cannulae with fine lumens are very difficult to clean
- Because disposable suction cannulae are recommended for very high risk procedures, especially neuro and ophthalmic

Characteristics

- Aspiration control hole has a diameter of 1,5 mm., allowing the surgeon a slower aspiration response. Whenever the hole is opened the aspiration on the tip will slowly decrease. Even with a very high aspiration pressure the cannula makes only a low amount of noise
- Equipped with a stylet in plastic material for cleaning the cannula in case of obstruction
- Complete line and specials can be easily produced following your indications
- Rounded edges, atraumatic distal end of lumen
- Non blocking design
- Strong enough to use as a probe
- Cannulae are made of Steel AISI 304
- Single angle
- Internal diameter is uniform over the entire length of the cannula
- Suction control handle made of copolymer styrene-butadiene (ABS)
- Matches all commonly used connection tubes
- Colour code for prompt recognition
- Latex free
- Sterilised by gamma radiation
- Single sterile blister packaging



* Product not in stock, available by specific request

| Code | Internal Diameter mm. | External Diameter mm. | CH | Cannula length mm. | Handle colour | Pcs. per BOX | Applications |
|-------------|-----------------------|-----------------------|----|--------------------|---------------|--------------|--|
| CSSR30S80* | 2,54 | 3,0 | 9 | 80 | BLUE | 60 | ENT, Plastics |
| CSSR30S130* | 2,54 | 3,0 | 9 | 130 | BLUE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |
| CSSR30S140* | 2,54 | 3,0 | 9 | 140 | BLUE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |
| CSSR30S160* | 2,54 | 3,0 | 9 | 160 | BLUE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |
| CSSR40S140* | 3,0 | 4,0 | 12 | 140 | WHITE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |
| CSSR40S160* | 3,0 | 4,0 | 12 | 160 | WHITE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |

Manufactured by:



Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy
Tel. +39 051.817902 +39 051.817850 Fax +39 051.817845
www.simitalia.it www.medeurope.eu
E-Mail: info@medeurope.eu

Distributed by:



SIM MICRO CANNULA “Slow Reaction” 90°L

Steel suction cannula rotated 90° to the left



Complete with suction control handle

Why?

- Because reusable suction cannulae with fine lumens are very difficult to clean
- Because disposable suction cannulae are recommended for very high risk procedures, especially neuro and ophthalmic

Characteristics

- Aspiration control hole has a diameter of 1,5 mm., allowing the surgeon a slower aspiration response. Whenever the hole is opened the aspiration on the tip will slowly decrease. Even with a very high aspiration pressure the cannula makes only a low amount of noise.
- Complete line and specials can be easily produced following your indications
- Rounded edges, atraumatic distal end of lumen
- Non blocking design
- Strong enough to use as a probe
- Cannulae are made of Steel AISI 304
- Single angle
- Internal diameter is uniform over the entire length of the cannula
- Angle and rotate to 90° in relation to the handle to facilitate a lateral hold
- Suction control handle made of copolymer styrene-butadiene (ABS)
- Matches all commonly used connection tubes
- Colour Code for prompt recognition
- Latex free
- Sterilised by gamma radiation
- Single sterile blister packaging

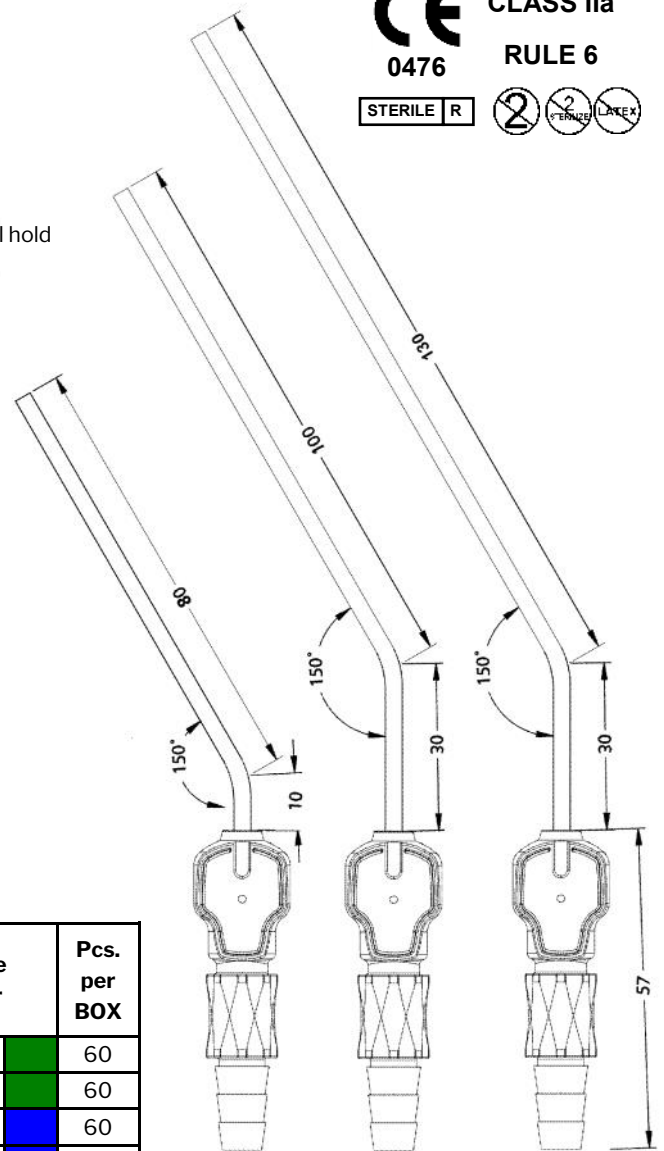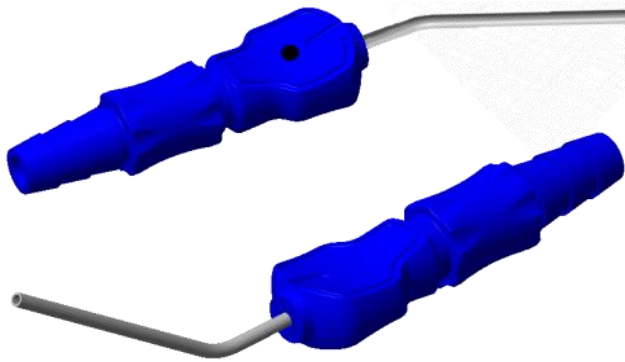

| Code | Int. Diameter mm. | Ext. Diameter mm. | CH | Cannula length mm. | Handle colour | Pcs. per BOX |
|------------|-------------------|-------------------|----|--------------------|---------------|--------------|
| CSSR20L130 | 1,45 | 2,0 | 6 | 130 | GREEN | 60 |
| CSSR20L160 | 1,45 | 2,0 | 6 | 160 | GREEN | 60 |
| CSSR30L130 | 2,54 | 3,0 | 9 | 130 | BLUE | 60 |
| CSSR30L160 | 2,54 | 3,0 | 9 | 160 | BLUE | 60 |
| CSSR40L130 | 3,0 | 4,0 | 12 | 130 | WHITE | 60 |
| CSSR40L160 | 3,0 | 4,0 | 12 | 160 | WHITE | 60 |

Manufactured by:




Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy
Tel. +39 051.817902 +39 051.817850 Fax +39 051.817845
www.simitalia.it www.medeurope.eu
E-Mail: info@medeurope.eu

Distributed by:



SIM MICRO CANNULA “Soft Touch”

Steel suction cannula with suction control handle and atraumatic silicone tip

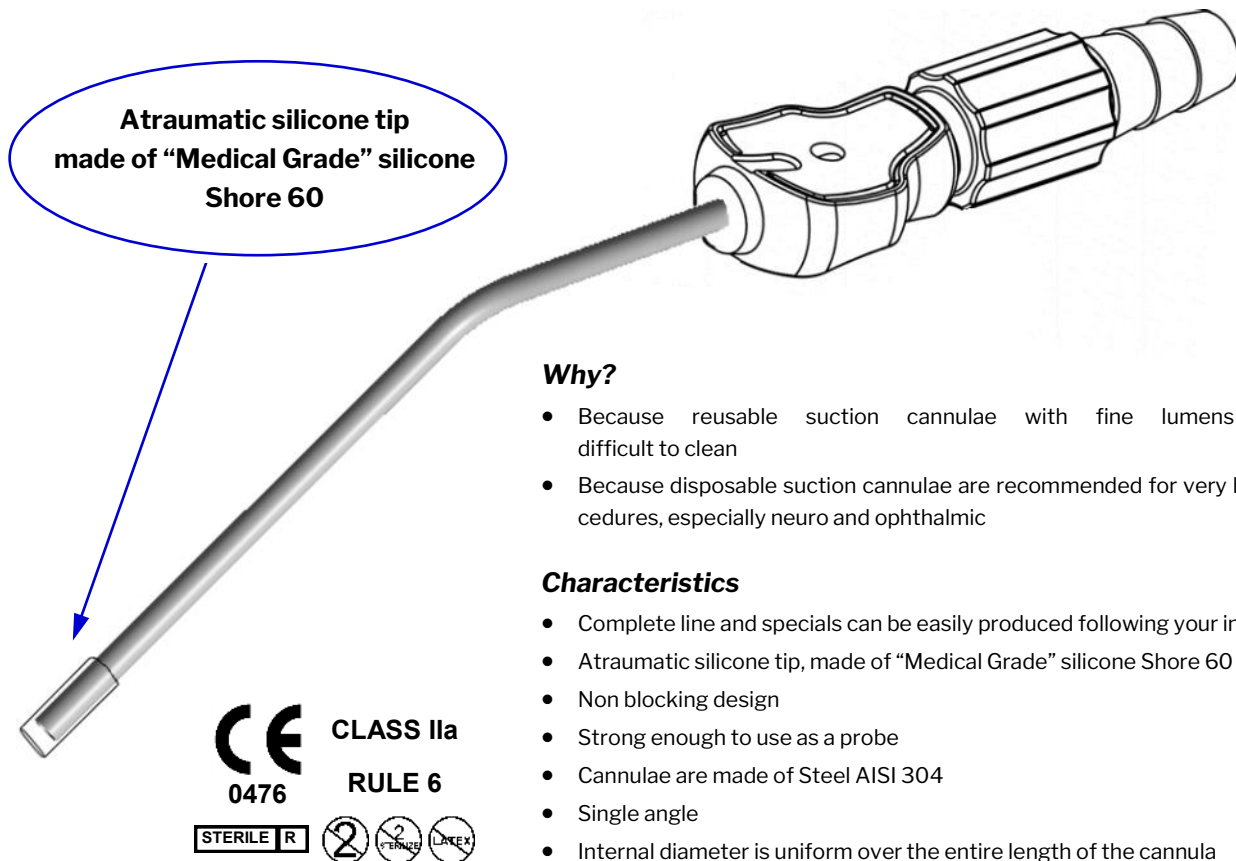

Why?

- Because reusable suction cannulae with fine lumens are very difficult to clean
- Because disposable suction cannulae are recommended for very high risk procedures, especially neuro and ophthalmic

Characteristics

- Complete line and specials can be easily produced following your indications
- Atraumatic silicone tip, made of “Medical Grade” silicone Shore 60
- Non blocking design
- Strong enough to use as a probe
- Cannulae are made of Steel AISI 304
- Single angle
- Internal diameter is uniform over the entire length of the cannula
- Suction control handle made of copolymer styrene-butadiene (ABS).
- Matches all commonly used connection tubes
- Colour Code for prompt recognition
- Latex free
- Sterilised by gamma radiation
- Single sterile blister packaging

| Code | Internal Diameter mm. | External Diameter mm. | CH | Cannula length mm. | Handle colour | Pcs. per BOX | Applications |
|-------------|-----------------------|-----------------------|----|--------------------|---------------|--------------|--|
| CSSR20X80S | 1,45 | 2,0 | 6 | 80 | GREEN | 60 | ENT, Plastics, Otological Surgery & outpatients |
| CSSR20X160S | 1,45 | 2,0 | 6 | 160 | GREEN | 60 | General, Plastics, Obs & Gynae, Paediatrics, Neuro, Oral & Max Fax |
| CSSR30X80S | 2,54 | 3,0 | 9 | 80 | BLUE | 60 | ENT, Plastics |
| CSSR30X160S | 2,54 | 3,0 | 9 | 160 | BLUE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |
| CSSR40X80S | 3,0 | 4,0 | 12 | 80 | WHITE | 60 | ENT, Plastics |
| CSSR40X160S | 3,0 | 4,0 | 12 | 160 | WHITE | 60 | General, Plastics, Paediatrics, Neuro, Oral & Max Fax, fine Orthopaedics |

Manufactured by:



Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy
Tel. +39 051.817902 +39 051.817850 Fax +39 051.817845
www.simitalia.it www.medeurope.eu
E-Mail: info@medeurope.eu

Distributed by:



SIM MICRO CANNULA

PVC aspiration cannula with suction control handle




CLASS IIa
RULE 6

STERILE R




- Particularly adapt for Neuro Surgery, ENT, Maxillo facial, dental surgeries and in general, Micro surgery
- Rounded, non-traumatic tips
- Cannulae are made of Medical PVC polyvinylchloride
- Double angle
- Internal diameter is uniform over the entire length of the cannula
- Aspiration control handle made of copolymer styrene-butadiene (ABS).
- Colour Code for prompt recognition
- Latex free
- Sterilised by gamma radiation
- Single sterile blister packaging

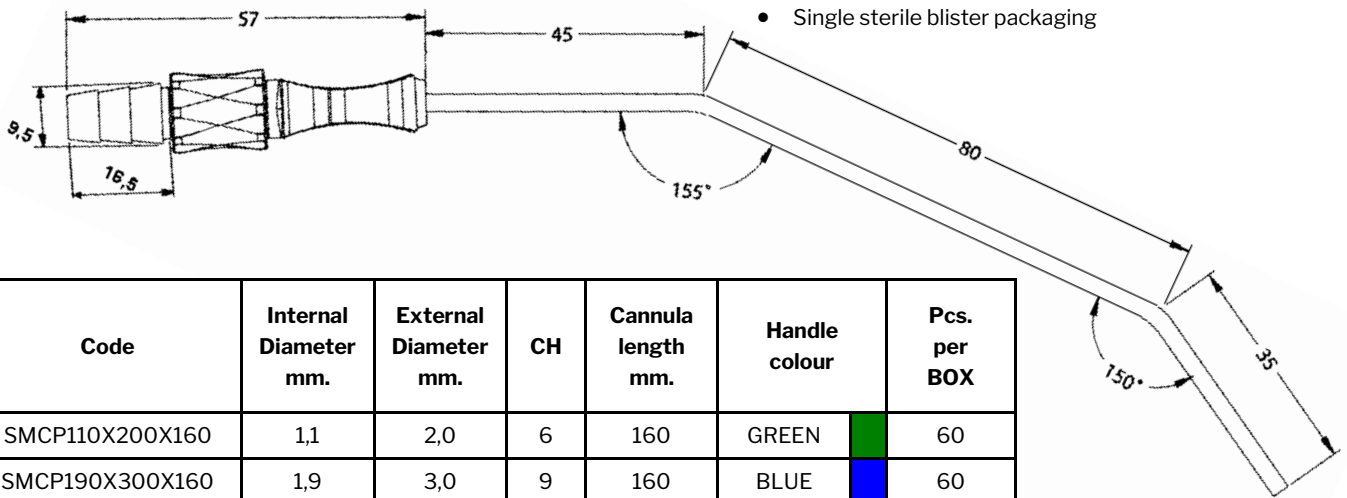

| Code | Internal Diameter mm. | External Diameter mm. | CH | Cannula length mm. | Handle colour | Pcs. per BOX |
|-----------------|-----------------------|-----------------------|----|--------------------|---------------|--------------|
| SMCP110X200X160 | 1,1 | 2,0 | 6 | 160 | GREEN | 60 |
| SMCP190X300X160 | 1,9 | 3,0 | 9 | 160 | BLUE | 60 |
| SMCP290X400X160 | 2,9 | 4,0 | 12 | 160 | WHITE | 60 |

SIM MICRO HANDLE

Aspiration control handle for needles / cannulae

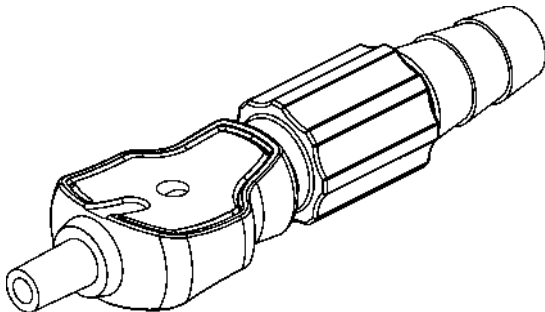

CLASS Is
RULE 2

STERILE R



- Better hold and manoeuvrability
- Complete with aspiration control
- Made of white copolymer styrene-butadiene (ABS)
- Latex free
- Single sterile blister packaged
- Sterilised by gamma radiation
- Non invasive

REF. SMH
Box 40 Pcs.

Manufactured by:



Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy
Tel. +39 051.817902 +39 051.817850 Fax +39 051.817845
www.simalia.it www.medeurope.eu
E-Mail: info@medeurope.eu


Distributed by:



SIM MICRO CANNULA for microlaryngoscopy

Steel suction cannula with suction control handle

- Aspiration control hole has a diameter of 1,5 mm., allowing the surgeon a slower aspiration response.
- Whenever the hole is opened the aspiration on the tip will slowly decrease. Even with a very high aspiration pressure the cannula makes only a low amount of noise.
- Particularly recommended for DIRECT SUSPENSION MICROLARYNGOSCOPY
- Complete line and specials can be easily produced following your indications
- Rounded edges, atraumatic distal end of lumen
- Non blocking design
- Strong enough to use as a probe
- Cannulae are made of Steel AISI 304
- Single angle
- Internal diameter is uniform over the entire length of the cannula
- Suction control handle made of copolymer styrene-butadiene (ABS).
- Matches all commonly used connection tubes
- Colour Code for prompt recognition
- Latex free
- Sterilised by gamma radiation
- Single sterile packaging

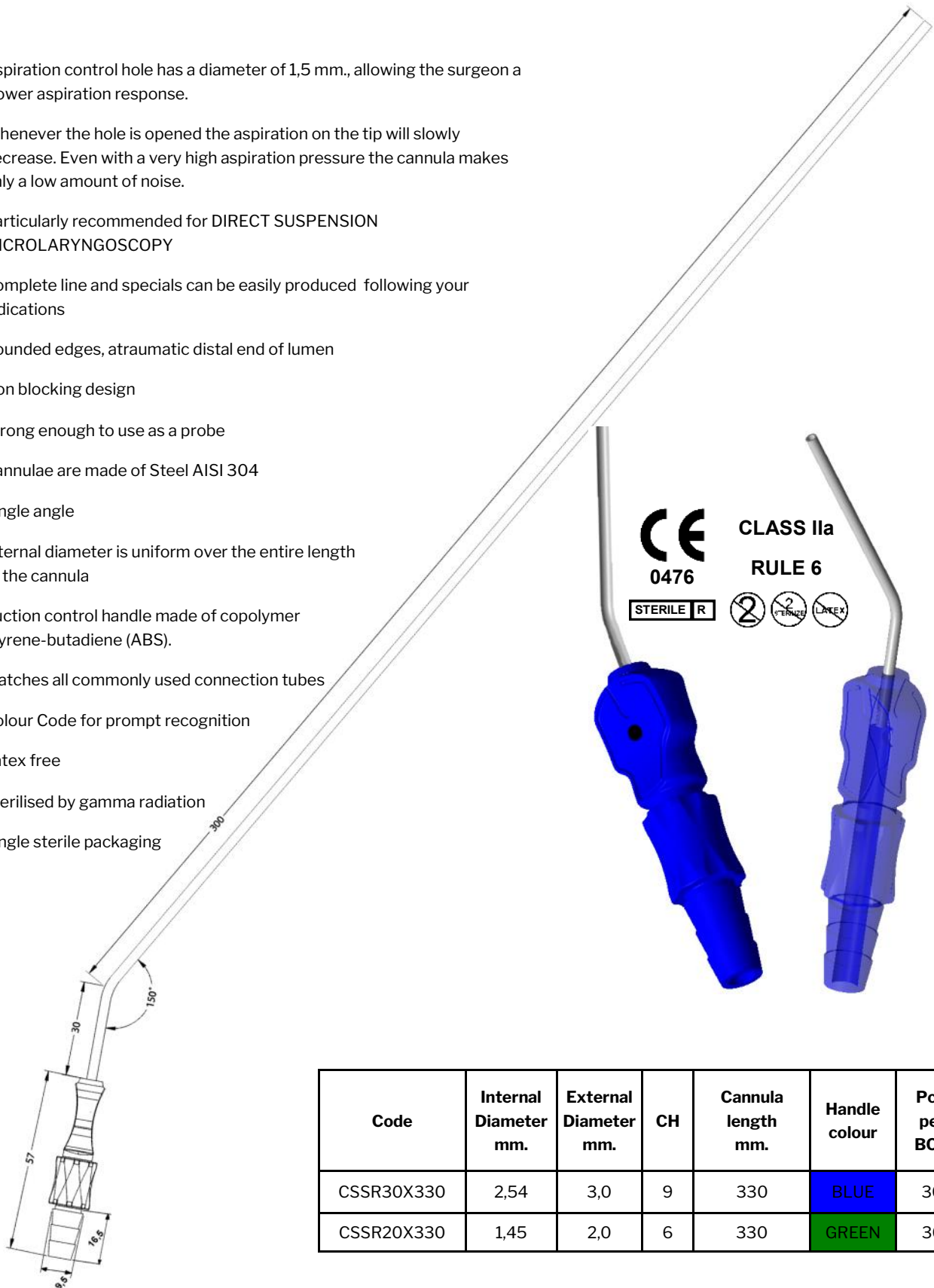

| Code | Internal Diameter mm. | External Diameter mm. | CH | Cannula length mm. | Handle colour | Pcs. per BOX |
|------------|-----------------------|-----------------------|----|--------------------|---------------|--------------|
| CSSR30X330 | 2,54 | 3,0 | 9 | 330 | BLUE | 30 |
| CSSR20X330 | 1,45 | 2,0 | 6 | 330 | GREEN | 30 |

Manufactured by:



Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy
 Tel. +39 051.817902 +39 051.817850 Fax +39 051.817845
 www.simitalia.it www.medeurope.eu
 E-Mail: info@medeurope.eu

Distributed by:

