


# NEW Safety Count System

Revision date: 18.04.2023

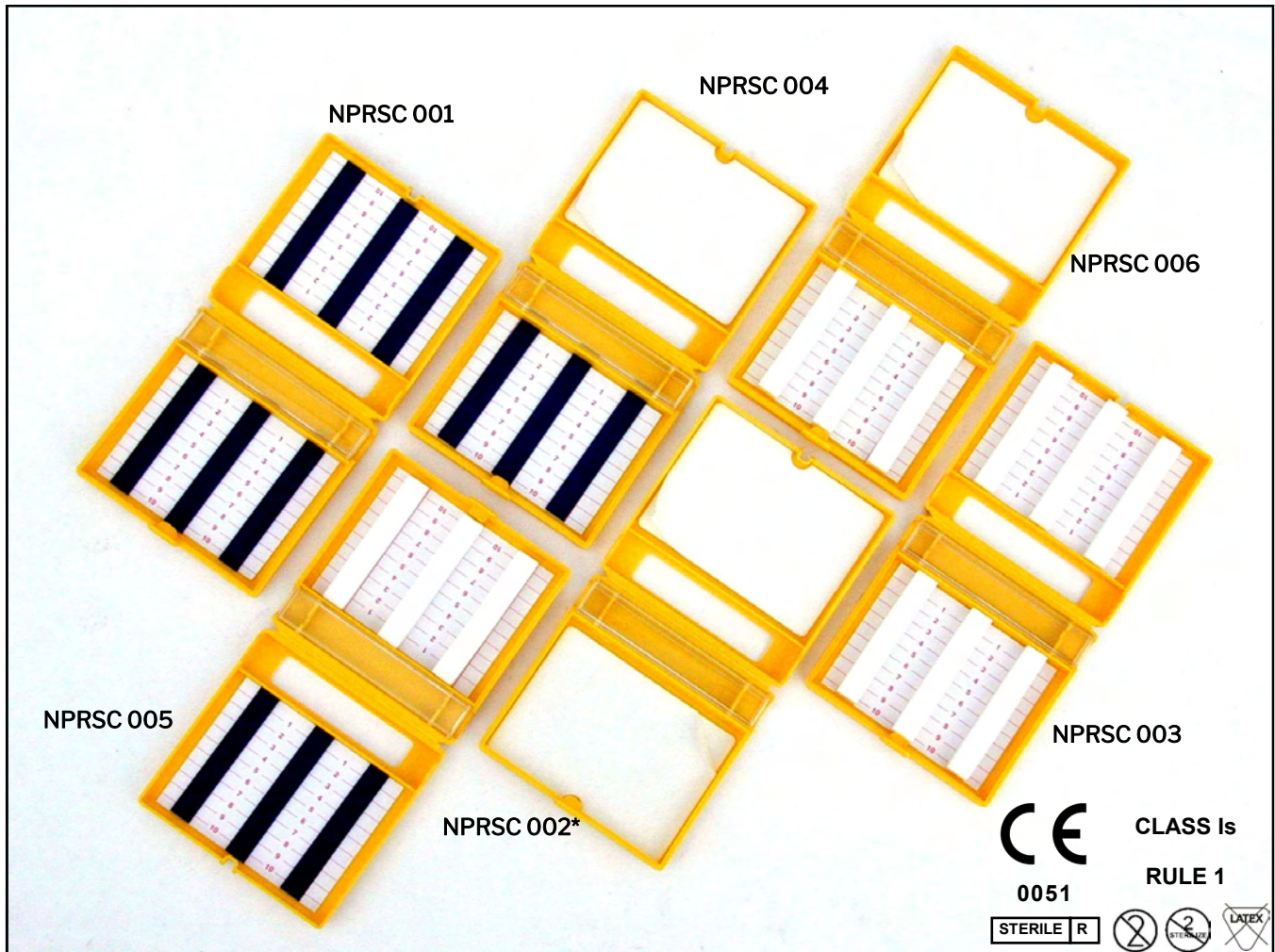

The SAFETY COUNT SYSTEM sharps counting and disposal system guarantees the safe elimination of scalpel blades and suture needles used in the operating room.

## Characteristics

Used sharps of all kinds can be counted and disposed of without the risk of contamination. The Safety Count System is safe, simple and economic.

- different models allow maximum user choice;
- the exclusive bilateral blade detachment system is efficient on a large variety of scalpel blades
- incorporates a resistant safety closure which protects against accidental opening (which can however be opened if necessary using the scalpel handle) ;
- strong, yellow, plastic "anti-puncture" container;
- sterile, blister packaged;
- conveniently packaged 18 pcs./dispenser box;
- sterilized by gamma rays.








## Codes

NPRSC 001: magnet + magnet  
NPRSC 002\*: adhesive + adhesive  
NPRSC 003: foam + foam  
NPRSC 004: adhesive + magnet  
NPRSC 005: foam + magnet  
NPRSC 006: foam + adhesive

Dispenser containing 18 Safety Count System units.  
108 pcs. per box

\* Product not in stock, available by specific request

PATENT PENDING

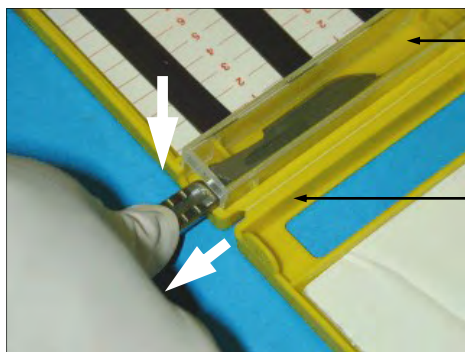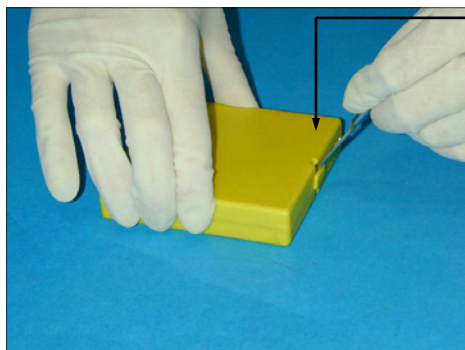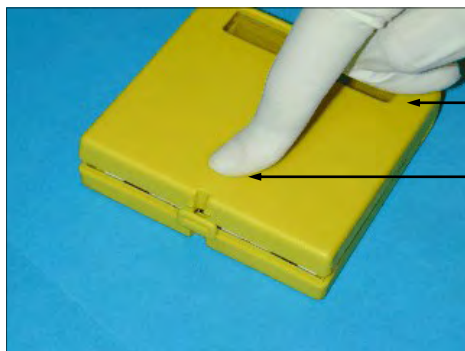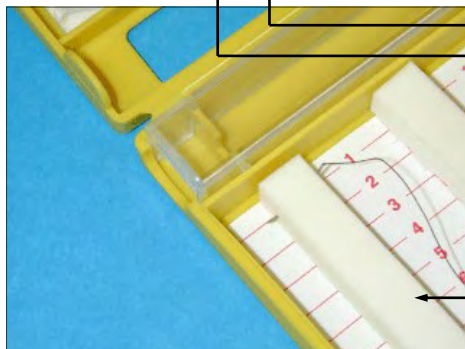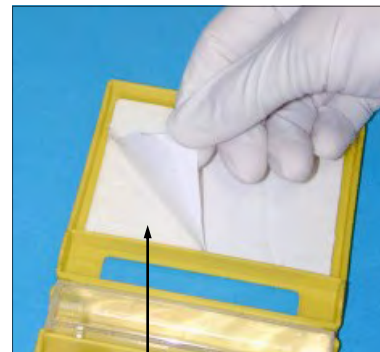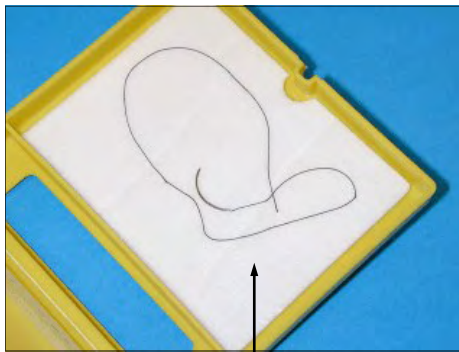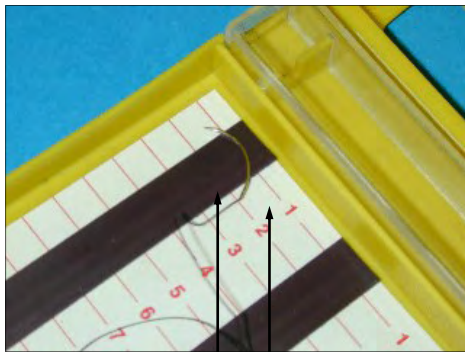

- Adhesive base which withholds contaminated materials
- Numbered base for precise counting
- Magnetic or foam strips that withhold used suture needles
- Strong, punctureproof yellow plastic container
- Safe click-locking closure which resists accidental opening, but which may be opened intentionally using the scalpel handle
- The double ended blade detachment device, provides easy and safe use for both right and left handed users
- Transparent cover
- After inserting the scalpel into the detachment device, push and pull back on the handle
- Self-fitting internal edges for increased closure safety
- Practical 18 piece dispenser

**CE**  
0051

STERILE R



**CLASS Is**  
**RULE 1**

**PATENT PENDING**



Manufactured by:



Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy  
Tel. +39 051.817902 +39 051.817850 Fax +39 051.817845  
www.simitalia.it www.medeurope.eu  
E-Mail: info@medeurope.eu

Distributed by:

