

Latex free pessary

Revision date: 03.06.2026



USE:

Pessary made of biocompatible Medical Grade platinum cured silicone for the containment of uterine prolapse.

MATERIALS:

Biocompatible 100% Medical Grade platinum cured silicone, transparent.
Shore hardness index: 60.

TECHNICAL CHARACTERISTICS:


- Maximum comfort due to the softness of the material and decreased weight (from 15.79 to 36.87 gr. based on the size);
- Maximum tolerability due to the medical grade silicone used;
- Maximum safety: the device is STERILE and for single use; Latex and Phthalate free;
- Maximum manoeuvrability by medical personnel responsible for insertion of the device: ready for use, packaged in single blister with 'peel' opening.

INSTRUCTIONS FOR USE:

- The device must be prescribed and inserted exclusively by specialised and certified medical personnel, ensuring sterility during application.
- The Device should be applied inside the osteo-vaginal inlet, bending the pessary ring elliptically to facilitate its entry and encircling the cervix. The pessary has a long-term usability, over 30 days, according to the criteria set out in Annex VIII of Regulation (EU) 745/2017.

STORAGE AND DISPOSAL:

- Store the device in a cool, dry place, away from direct light and from heat sources.
- Attention: the device is STERILE. Verify the condition of the packaging before use: in the event of damage do not use.
- After use, dispose of the device according to the laws on hospital devices.

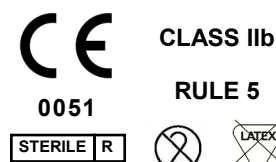

REF	External Diameter
A982750578	50 mm
A982750592	55 mm
A930244734	60 mm
A930244761	65 mm
A930244785	70 mm
A930244797	75 mm
A930244811	80 mm
A930244823	85 mm
A930244672	90 mm
A982750604	95 mm
A982750616	100 mm

Manufactured by:



Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy
Tel. +39 051.817902 +39 051.817850
www.simitalia.it www.medeurope.eu
E-Mail: info@medeurope.eu

Distributed by:

