


GASTRIC BAND

Dual Component Band

Ring For Bariatric Surgery

Revision date: 16.07.2026

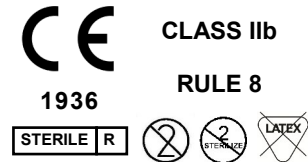

Implantable silicone ring used in bariatric surgery procedures, such as banded sleeve gastrectomy, gastric bypass, and banded mini-gastric bypass. It can be implanted laparoscopically through a 12 mm trocar or laparotomy. The device is indicated for both primary and revision surgery. It is preformed to facilitate placement around the stomach. A stylet at one end of the device has a bulge at the base, which allows the device to close properly, mating with a hole at the opposite end of the device. The hole is located at the opposite end of the device from the closing stylet. It engages with the rounded base of the closing stylet, which, being deformable, allows entry into the hole.

Characteristics

- Available in 4 sizes: Ø 5cm, Ø 6cm, Ø 7cm, Ø 8cm.
- Material: medical grade implantable silicone, E-PTFE clamp, white color, x-ray, shore: 20A.
- Sterilised by gamma rays.
- Double packaging (internal blister and external pouch).
- Latex free.

Codes


REF **DCB-05**

REF **DCB-06**

REF **DCB-07**

REF **DCB-08**

Packaging: 1 piece/box

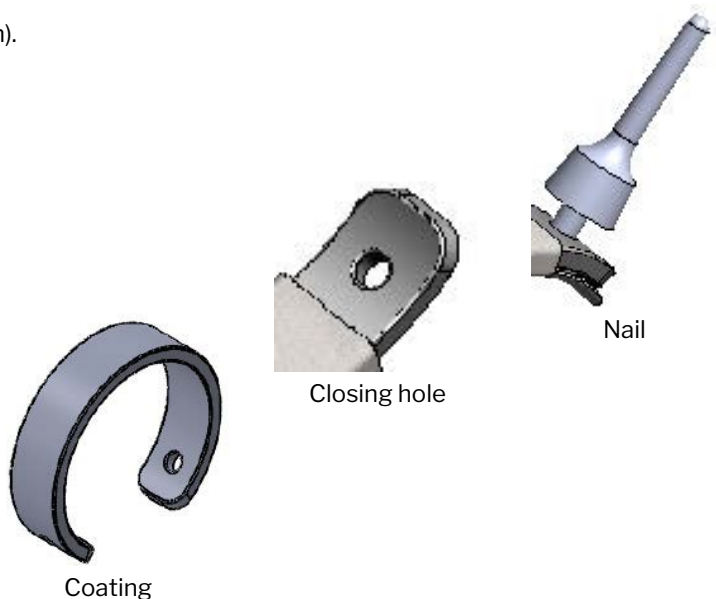

Manufactured by:



Via Gramsci, 9 - 40018 San Pietro in Casale (BO) - Italy
Tel. +39 051.817902 +39 051.817850
www.simitalia.it www.medeurope.eu
E-Mail: info@medeurope.eu

Distributed by:

